


INFANT

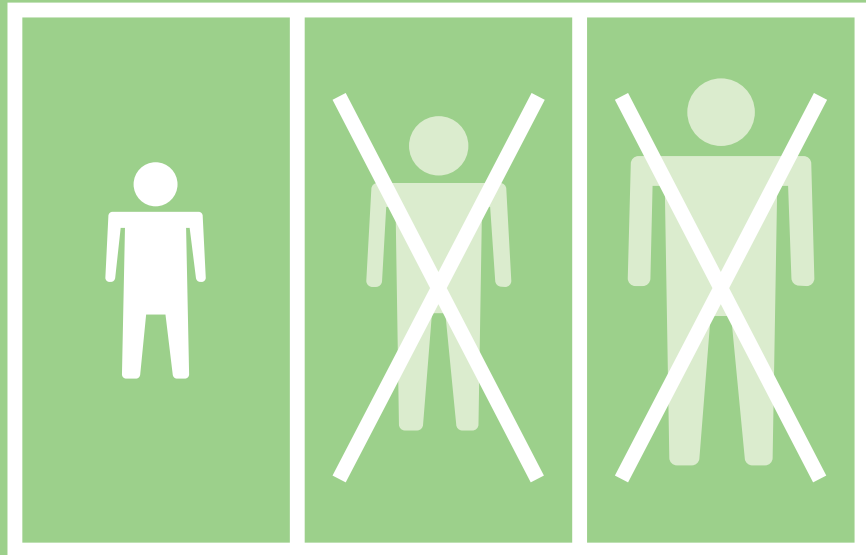

Ambu[®] **SPUR[®] II**
Single Patient Use Resuscitator

RESPIRATORY CARE

PRECAUTIONS:

Unpack the resuscitator and do the following before placing it ready for use:

- Unpack the resuscitator from the carrying bag.
- Attach the oxygen supply tube (if not attached already) and make a functional check as prescribed in the directions for use.
- Pack the resuscitator in the carrying bag (if not done already) with face masks, airway and other ancillary devices prescribed for the intended location of use.
- Make sure that the resuscitator is never stored in a deformed state.
- Users of this product must be informed of the important information in the directions for use.
- Non-sterile.
- Storage temperature: -40°F (-40°C) to +140°F (+60°C).
- Operating temperature: -4°F (-20°C) to +122°F (+50°C).
- The integrity of kits issued for storage ready for use should be inspected at the interval established by your local protocol.

CAUTION:

U.S. Federal law restricts this device to sale by or on order of a physician.

WARNING:

- Never excessively squeeze the bag during storage. Otherwise permanent deformation of the bag will occur which may reduce the ventilation efficiency.



Manufacturer:
Ambu A/S
Baltorpbakken 13
DK-2750 Ballerup
Denmark
Phone +45 72 25 20 00
Fax +45 72 25 20 50
ambu@ambu.com
www.ambu.com

Subsidiary:
Ambu Inc.
6740 Baymeadow Drive
Glen Burnie, MD 21060
USA
Phone (410) 768-6464
Customer service 1-800 ambu inc.
Fax 1-800 ambu ord